

## Deliver with confidence



### Maximize delivery performance with advanced technology

To deliver an agent as sophisticated and uniquely controllable as Desflurane, you need a vaporizer to match—the Tec 6 Plus Vaporizer from Datex-Ohmeda. The Tec 6 Plus Vaporizer, an exceptional vaporizer for the delivery of an exceptional agent.

- The Tec 6 Plus is an electronic vaporizer that heats Desflurane to maintain constant temperature and vapor pressure for consistent output
- To enhance the titratability and depth regulation benefits of Desflurane, the Tec 6 Plus provides a concentration range of 1% to 18%
- Planned factory service free, helping to reduce overall life cycle cost
- For convenience and safety, the Tec 6 Plus LED display indicates vaporizer status, including:
  - Operational
  - No output
  - Low agent
  - Warm-up
  - Alarm battery low
- Attaches to Datex-Ohmeda's interlocking Selectatec® manifold which allows you to change vaporizer arrangement from case to case or move vaporizers from suite to suite without the need for tools


# Tec 6 Plus Vaporizer



The Tec 6 Plus interlocks in series with Tec 5 and Tec 4 Vaporizers on the Datex-Ohmeda Selectatec® backbar

## Designed for safety

- The Tec 6 Plus features several intrinsic vaporizer monitoring functions and alarms that constantly monitor vaporizer status.
- It incorporates a self-test system which checks a range of conditions each time the unit is turned on.
- Once in operation, an LCD display clearly indicates agent quantity. If the level of agent in the sump drops to 60 ml, an alarm will sound.
- The no-output alarm has a battery backup to help maintain alert status even if power is lost. A distinctive audio tone will sound when vaporizer can no longer deliver agent because AC power is disconnected.
- The Tec 6 Plus will even recognize if it has been tilted during use and indicates that output has been shut off.
- Improved tonal quality for alarm audio sounds while conveying alarm importance in alarm sound sequence, visual alarm color and flash rate.

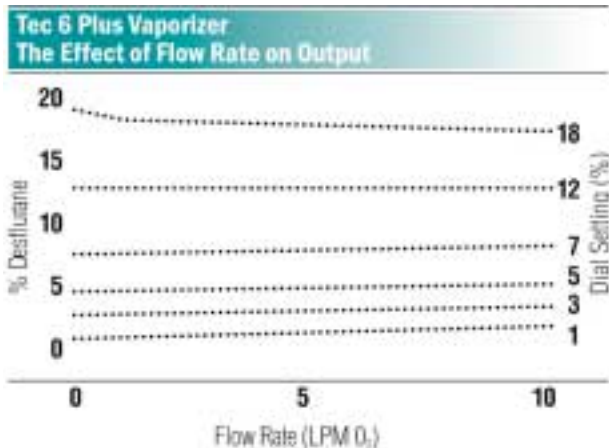

**Datex-Ohmeda Division** • Instrumentarium Corp.  
P.O. Box 900, FIN-00031 Datex-Ohmeda, Finland  
Tel. +358 10 39411 • Fax +358 9 146 3310

[www.datex-ohmeda.com](http://www.datex-ohmeda.com)

## Added flexibility

- The Tec 6 Plus can be mounted on any anesthesia machine equipped with a Datex-Ohmeda Selectatec manifold. With the Selectatec®, the Tec 6 Plus can interlock in series with Tec 5 and Tec 4 Vaporizers. There are specific models of the Tec 6 Plus available for use on Narkomed®\* and Dräger anesthesia machines.

## Large capacity, easy refilling

- The Tec 6 Plus has a large 425 ml sump capacity to accommodate nearly two full bottles of Desflurane. This lets you use the vaporizer for extended periods before refilling. In addition, refilling can be done while the vaporizer is in use.
- The exclusive Saf-T-Fill valve of the Desflurane bottle is designed to fit only the complementary filling device of the Tec 6 Plus so you can refill quickly and safely. This valve helps prevent accidental release of vapor into the OR and ensures that only Desflurane is introduced into the Tec 6 Plus.

## Tec 6 Plus warranty and maintenance

- The Tec 6 Plus Vaporizer comes with a two-year warranty
- **Planned factory service free**

For complete user maintenance requirements, refer to operations manual.

For more information on how the Tec 6 Plus Vaporizer can benefit you, contact your local Datex-Ohmeda representative. Or call Datex-Ohmeda at **1 800 345 2700**. Outside the U.S., call 1 608 221 1551.

\* Narkomed® is a registered trademark of North American Dräger.

